# Little Anne - AED Little Anne

ENGLISH

Directions for Use

ESPAÑOL

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## Little Anne

Little Anne has been developed to provide a cost effective alternative for CPR training programs. More manikins can be used in the classroom, increasing hands-on practice and active participation for the students. All necessary educational requirements have been maintained to assure quality training.

Each Little Anne model simulates an adult of average physiology and anatomy and is designed for realistic training of basic life support skills, primarily cardiopulmonary resuscitation (CPR).

Features included are:

- Natural obstruction of the airway
- Realistic facial features and a movable jaw
- Realistic compliance for ventilations and compressions
- Chest rise and fall with proper ventilations
- Realistic landmarks for compression point location
- Audible confirmation of correct compression depth
- Carrying bag opens into a training mat.

## AED Little Anne

The AED Little Anne was developed to work in combination with the Laerdal or Philips brand AED Trainer 2 device as a complete training system. Laerdal Link Technology in the AED Little Anne Training System, requires the student to properly place the training pads on the manikin chest before the AED Trainer 2 will proceed through the selected scenario. If pads are improperly placed, the AED Trainer 2 will respond with the appropriate voice prompts. This is accomplished with no visible connectors on the chest skin. The AED Little Anne also has LEDs in the chest skin indicating the center of correct pad placement. The LEDs light up automatically when pads are attached correctly. If an electrode is loose or incorrectly placed on the manikin, the AED Trainer 2 will repeat the voice prompt to apply the electrode pads.

*Note: For use with the AED Little Anne manikin with Laerdal Link Technology, electrode training pads with a white connector must be used (these electrodes are also marked with the Laerdal Link Technology logo).*

Additional features for AED Little Anne are:

- Sensors in the chest skin detect proper pad placement
- Pad placement feedback provided by LEDs in the chest skin.

## Instructions for use

Remove the carrying bag from the plastic bag. Open the bag and lay it flat on the floor. This will now serve as a training mat. Little Anne is delivered fully assembled and ready for use.

### Unpacking

Some additional supplies are provided for your convenience. Included with your Little Anne / AED Little Anne manikin (cat. No. 02 00 20, 02 00 25 or 02 00 50) are:

- Little Anne with Compression Clicker
- 1 extra Resusci Manikin Face
- 1 extra Little Anne Airway
- Carrying Bag/Training Mat
- Hygiene solution for training manikins
- Directions for Use
- Jacket (AED Little Anne only)

### Using the compression clicker

Little Anne is supplied with a compression clicker that will confirm correct compression depth. When the clicker is "on", the student will hear a "click" when the torso is compressed 1 1/2 inches (38 mm).

Fig. 1

To use the clicker:

The manikin is delivered with the clicker in the "on" (inner) position.

Fig. 2

To disable the clicker:

Pull out the clicker lever.

Note:

Always push the clicker lever to the "On"-position before repacking the manikin.

Fig. 1


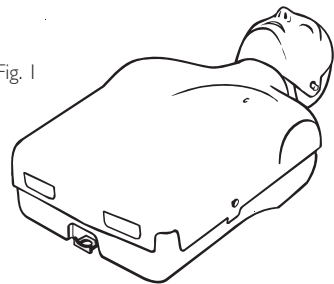
Fig. 2

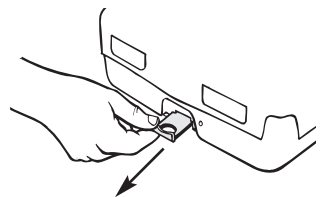
## Sanitation

Sanitation of Manikin Faces, see separate sheet.

### Little Anne Airway

The Little Anne Airway is designed for disposal after each class.

Fig. 3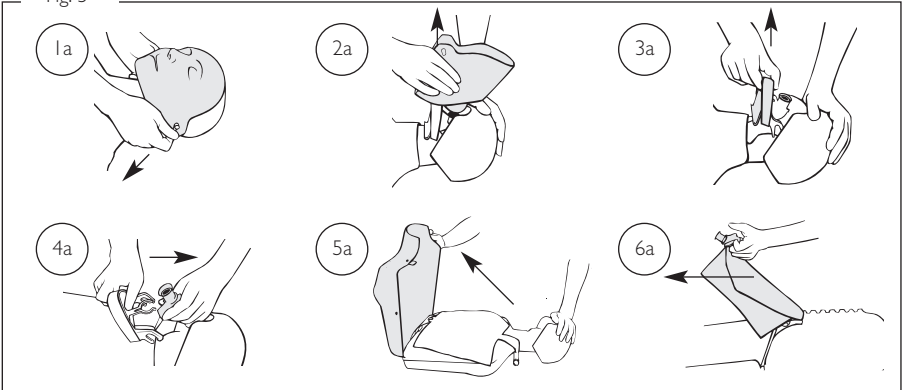


To remove Fig. 3

- 1a Detach manikin face at each ear.
- 2a Lift manikin face off valve coupling.
- 3a Remove jaw from posts in head.
- 4a Remove valve.
- 5a Open chest cover.
- 6a Remove used airway and discard.

To install Fig. 4

- 1b Lay lung flat on torso and attach to hook.
- 2b Snap airway valve onto jaw.
- 3b Reattach jaw to posts in head.
- 4b Fasten chest cover onto the four retainers.
- 5b Push face coupling into airway valve.
- 6b Fasten face onto retainers at each ear.

Fig. 4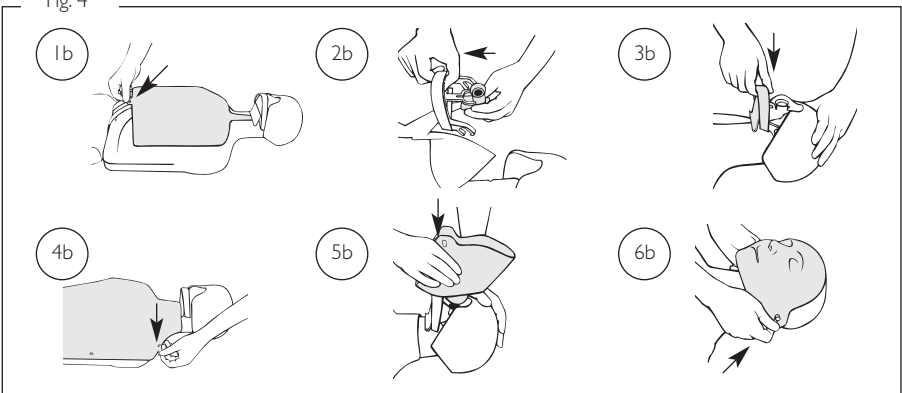


## Maintenance

**General manikin cleaning.** Routine disinfection of the entire manikin is not required, but periodic cleaning of the head, torso, chest skin, and training mat is recommended. These should be washed with a luke warm soapy solution, and then rinsed with a clean damp cloth. Proper care will maintain the appearance and general condition of your Little Anne.

### To change clicker

Fig. 5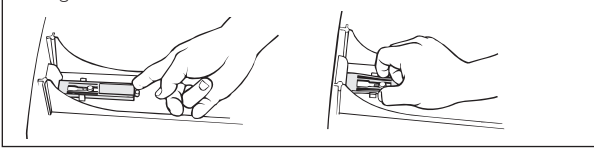


To remove - Fig. 5

Open the chest cover.  
Remove the rib plate.  
Set the clicker in the "off"  
position (outer position).

Press together the sides of the  
clicker and lift it up. Push it out  
through the opening in the  
torso.

Fig. 6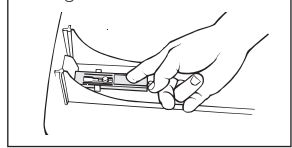


To install - Fig. 6

Insert the clicker from outside  
(with the pad facing upward).  
Press the clicker down until it  
snaps into place.

### To change clicker activator - Fig. 7

The clicker activator is located underneath the rib plate. It is fastened by snapping it in place over the edge and into the two holes of the rib plate as illustrated.

Fig. 7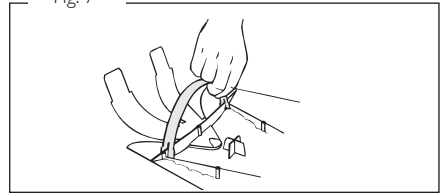


### To change chest skin - Fig. 8

Unhook the chest skin from its retainers.

*Note: For the AED Little Anne model be aware of wires connecting the chest skin to the battery box under the rib plate. Disconnect the two wires from the battery box (when connecting the new chest skin wires to the battery box it does not matter which wire goes to which socket).*

Fig. 8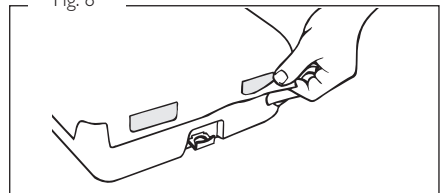

### To change head - Fig. 9

#### To remove


When face, airway and jaw have been removed (described in "Fig. 3"), flex the lower edges of the head outward until the head can be removed.

#### To install

Grasp the manikin head with both hands, with your fingers on the inside of the head. Gently pull outwards and slide the head onto the neck posts, one side at a time.

*To install the airway, face and jaw, see page 4.*

Fig. 9

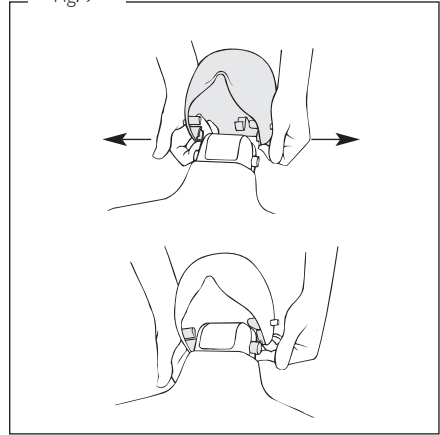## AED Little Anne only

### To change battery

AED Little Anne comes with a standard alkaline 6LF22 / 6LR61 9 Volt battery.

- Open the chest skin.

*Note: Be aware of wires connecting the chest skin to the battery box under the rib plate.*

- Remove the old battery and install a new 9 Volt alkaline battery. Be sure to follow the positive (+) and negative (-) battery icons etched into the inside of the battery holder when inserting new battery.

### Battery low indicators

- Depressing the test button on the battery board will illuminate the LEDs in the chest skin if battery strength is sufficient for operation. If the LEDs do not light when the button is depressed a new battery is required.
- A "Battery low" voice prompt from the AED Trainer 2 will be heard immediately after connecting Laerdal Link training pads if the manikin battery is low.

## Parts

### Models

02 00 20	Little Anne
02 00 25	Little Anne, w/dark skin
02 00 50	AED Little Anne

### Parts

02 01 10	Back section
02 01 20	Rib plate
02 10 00	Chest cover
02 10 05	Chest cover w/dark skin
02 05 10	Compression spring
02 01 11	Compression clicker
02 01 12	Clicker activator
02 02 00	Head complete
02 02 01	Jaw assembly
02 02 02	Head assembly
02 06 10	Directions for Use
02 07 00	Carrying bag

### Accessories

02 03 00	Airways complete, pkg. of 24
02 03 01	Airways complete, pkg. of 96
02 03 10	Valves complete, pkg. of 50
02 03 11	Filter & Lung, pkg. of 100
31 02 10	Manikin Faces, pkg. of 6
31 02 15	Manikin Faces, dark skin, pkg. of 6
15 42 30	Virkon Rapid Pack (10x5g tablets and bottle)
15 42 31	Virkon tablets 10x5g
15 24 00	Resusci Manikin Wipes, pkg. of 50
15 24 01	Resusci Manikin Wipes, pkg. of 1200
15 12 01	Manikin Face Shields, 6 rolls of 36 ea.

### Additional parts for AED

#### Little Anne only:

02 50 10	Battery board
02 50 15	AED Chest skin
02 50 30	Soft pack, AED Little Anne, Blue
02 50 20	Jacket, AED Little Anne
94 50 90	Training Pads, Laerdal Link Training System

For customers of Philips Medical Systems, please contact your sales representative or call 1 (800) 263-3342 for ordering information.

Product specifications are subject to change without notice.

This product is in compliance with the essential requirements of Council Directive 89 / 336 / EEC; EMC Directive.

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## After class


### Disinfection: Preparing the Resusci Manikin Face for reuse

After class, a used manikin face can be disposed of or reused.

Note: Wear rubber gloves\* during cleaning.

To prepare for reuse:

1. Remove face and detach the coupling from inside the face.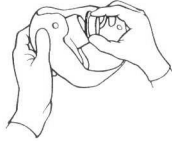

2. Clean and disinfect faces.
  - a) Wash with soapy warm water - scrub vigorously to remove dirt from interior surfaces.
  - b) Rinse with clean water to remove soap residue.

- c) Disinfecting solution - to disinfect the manikin face. (A sodium hypochlorite solution prepared for single class use, is an effective and recommended disinfectant. This solution should have at least 500 ppm free available chlorine - 1 part liquid household bleach to 100 parts tap water.
- d) Rinse with clean water to remove disinfecting solution residues.

3. Allow parts to dry.

4. Reassemble face coupling and face skin.

5. Remount face on manikin after reinstallation of airway or put in a clean bag for future use.


\* Do not use latex gloves as this may stain the face mask.

### Little Anne Airway

The Little Anne Airway is designed for disposal after each class

#### To remove

1. Detach face mask at each ear.
2. Lift face mask off valve coupling.
3. Remove jaw from posts in head.
4. Remove valve.
5. Open chest cover.
6. Remove used airway and discard.

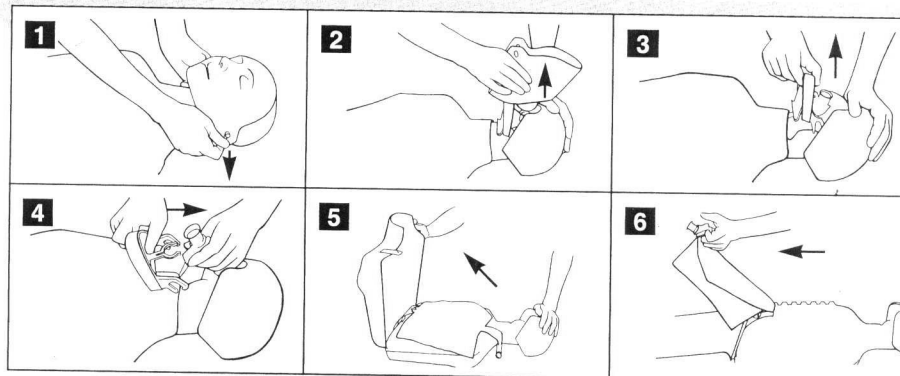**Laerdal**

*helping save lives*