

Simulated Smallpox Module Set

Directions for Use



Laerdal[™]
helping save lives

Items Included:

- Inoculation Injection Pad
- Smallpox Normal Reaction
- Smallpox Abnormal Reaction

Skills Taught:

- Offers the ability to identify the effects of abnormal or normal skin reactions related to smallpox exposure

These modules can be adapted with many Laerdal manikins that use deltoid injection pads, such as SimMan, MegaCode Kelly and most patient care manikins.



276-16350



276-16250



276-16150

To use modules with Laerdal manikin, replace injection pads with Smallpox Modules.

- a. To remove, squeeze pad with fingers and pull.
- b. To replace, squeeze pad and insert in opening.

Replacement Parts:

Inoculation Injection Pad	276-16350
Smallpox Normal Reaction	276-16250
Smallpox Abnormal Reaction	276-16150

© Medical Plastics Laboratory, Inc
Document #1005306 rev A

February 19, 2003