

Suspected Coronavirus (COVID-19)

Section 1: Case Summary

Scenario Title:	Suspected Coronavirus (COVID-19) w Respiratory Failure
Keywords:	Infectious disease, provider safety, airway management
Brief Description of Case:	Case designed during the January 2020 COVID-19 outbreak in order to assess and improve team preparedness for safely and effectively caring for a critically ill coronavirus patient from triage through to intubation.

Goals and Objectives	
Educational Goal:	Practice personal and team safety while assessing and providing care to a patient with a potentially airborne respiratory illness requiring full personal protective equipment.
Objectives: (Medical and CRM)	1. Effective team communication from triage to intubation of a high-risk coronavirus patient 2. Mitigating exposure of HCPs, patients, by appropriately isolating high risk patients in negative pressure room 3. Mitigating personal risk by utilizing the appropriate donning/doffing of PPE 4. Conducting effective high acuity clinical care (i.e. intubation) in space and personnel limited negative pressure room effectively
Infection Prevention and Control Objectives (See Appendix C for details)	<i>The guidelines for the novel coronavirus are changing frequently as we receive new information about the virus. Infection prevention and control (IPAC) considerations also vary between institutions. Please review the most up-to-date guidelines and discuss with your IPAC team before running the simulation.</i>
EPAs Assessed:	Not for routine education

Learners, Setting and Personnel	
Target Learners:	☐ Junior Learners ☐ Senior Learners ☒ Staff
	☒ Physicians ☒ Nurses ☒ RTs ☒ Inter-professional
	☒ Other Learners: Infection Prevention and Control Team
Location:	☐ Sim Lab ☒ In Situ ☐ Other:
Recommended Number of Facilitators:	Instructors: 1
	Confederates: 1
	Sim Techs: 1

Scenario Development	
Date of Development:	January 26, 2020
Scenario Developer(s):	Dr. Alia Dharamsi, Dr. Soojin Yi, Dr. Kate Hayman
Affiliations/Institutions(s):	University of Toronto
Contact E-mail:	alia.dharamsi@gmail.com
Twitter:	Please credit the authors: @alia_dh @soojinder @hayman_kate when posting about use of this case on social media
Revised By:	
Version Number:	1



Suspected Coronavirus (COVID-19)

Section 2A: Initial Patient Information

A. Patient Chart					
Patient Name: Ms. Grace Yi		Age: 35		Gender: F	Weight: 60kg
Presenting complaint: Shortness of breath, cough, fever					
Temp: 39.2	HR: 140	BP: 100/60	RR: 22	O ₂ Sat: 90%	FiO ₂ : RA
Cap glucose: 7.1			GCS: 15		
Triage note: 35-year-old woman became febrile last night with coryza and woke up acutely short of breath with productive cough, rhinorrhea, and a subjective fever. She screens positive for potential coronavirus exposure due to fever, respiratory symptoms and a high-risk travel history.					
Allergies: None					
Past Medical History: None			Current Medications: Ibuprofen Acetaminophen		

Section 2B: Extra Patient Information

A. Further History	
<i>Include any relevant history not included in triage note above. What information will only be given to learners if they ask? Who will provide this information (mannequin's voice, confederate, SP, etc.)?</i>	
History per triage note. Additional travel history given.	

B. Physical Exam	
<i>List any pertinent positive and negative findings</i>	
Cardio: Tachycardia	Neuro: Nil
Resp: Crepitus and expiratory wheezes bilaterally, productive cough	Head & Neck: Coryza
Abdo: Nil	MSK/skin: Flushed
Other: Nil	



Suspected Coronavirus (COVID-19)

Section 3: Technical Requirements/Room Vision

A. Patient
☒ Mannequin: Adult
☒ Standardized Patient
☐ Task Trainer
☐ Hybrid
B. Special Equipment Required
Negative pressure/isolation room Airborne PPE for all involved Donning/doffing area/ ante room (either actual ante room if done in negative pressure room, or tape on the floor to designate mock ante room)
C. Required Medications
Routine drugs for intubation, sedation, paralysis
D. Moulage
Aerosolizer for creating secretions from mannequin Glo Dust on mannequin's face and upper body/arms See here for further instructions: https://drive.google.com/file/d/1sZsvL9mdML8Z0hv09p8zAaC0JizeZm9F/view?usp=sharing Credit to simulationist Roger Chow NB: this is a live document and may undergo updates
E. Monitors at Case Onset
☐ Patient on monitor with vitals displayed ☒ Patient not yet on monitor
F. Patient Reactions and Exam
<i>Include any relevant physical exam findings that require mannequin programming or cues from patient (e.g. – abnormal breath sounds, moaning when RUQ palpated, etc.) May be helpful to frame in ABCDE format.</i> A - URTI, cough, patent airway B - Bilateral crepitus, decreased air entry bilaterally due to shortness of breath, scant expiratory wheeze, no pedal edema or calf tenderness C - Tachycardic and febrile, D - Moving x4, GCS 15



Suspected Coronavirus (COVID-19)

Section 4: Confederates and Standardized Patients

Confederate and Standardized Patient Roles and Scripts	
Role	Description of role, expected behavior, and key moments to intervene/prompt learners. Include any script required (including conveying patient information if patient is unable)
Standardized Patient	Standardized patient (SP) is to arrive at triage coughing and febrile, with rapid respiratory rate and short of breath. SP is to describe travel history: was in mainland China last week for business. Visited Hubei Province. She returned two days ago and became ill 24 hours after landing in Canada. SP is to be too weak to walk Once SP is placed into negative pressure room, SP replaced with mannequin



Suspected Coronavirus (COVID-19)

Section 5: Scenario Progression

Scenario States, Modifiers and Triggers				
Patient State/Vitals	Patient Status	Learner Actions, Modifiers & Triggers to Move to Next State		Facilitator Notes
1. Triage Rhythm: Sinus tach HR: 140 BP: 100/60 RR: 22 O ₂ SAT: 90% T: 39.2°C GCS: 15	Short of breath, coughing, febrile Describes travel history	<u>Expected Learner Actions</u> ☐ Obtain vital signs ☐ Apply PPE to patient ☐ Move to neg pressure room ☐ Triage RN to contact MD and bedside team with details ☐ Call IPAC (or local equivalent) ☐ HCPs don appropriate PPE	<u>Modifiers</u> <i>Changes to patient condition based on learner action</i> <u>Triggers</u> <i>For progression to next state</i> - Move patient to neg pressure room, handover to bedside team	
2. Initial Assessment RR: 28 O ₂ SAT: 84%	Mannequin replaces SP	<u>Expected Learner Actions</u> ☐ IV Access, monitors ☐ Apply O ₂ by NRB mask ☐ Portable Xray, Labs, ECG ☐ Point of care ultrasound ☐ Call RT ☐ IV Bolus, consider antibiotics (empiric), swabs for flu and coronavirus, septic workup	<u>Modifiers</u> - O ₂ Sat increases to 90% with supplemental O ₂ <u>Triggers</u> - All action complete	
3. Intubation RR: 30 O ₂ SAT: 86% NRB	Patient becoming more hypoxic, agitated	<u>Expected Learner Actions</u> ☐ Prepare for intubation (push dose pressors, equipment, personnel, medications) ☐ Intubate	<u>Modifiers</u> <u>Triggers</u> - Successful intubation	Not anticipated to be difficult airway, easy intubation
4. Disposition RR: 14 O ₂ SAT: 92% (Vented 100% FiO ₂)		<u>Expected Learner Actions</u> ☐ Continue bolus fluids ☐ Routine post-intubation care ☐ Sedation ☐ Call ICU	<u>Modifiers</u> <u>Triggers</u> - Handover to ICU	



Suspected Coronavirus (COVID-19)

5. Exposure Management	Triage nurse to ask team lead what they should do now that they are exposed to potential case	☐ Identify potential exposed persons, nature of exposure ☐ Liaise with IPAC (or local equivalent) ☐ Isolate potential exposed persons until further IPAC instruction	<u>Modifiers</u> <u>Triggers</u> - All actions complete	
6. Aerosol Assessment (optional)		☐ Use black light to see where Glo Germ lights up on care providers and equipment to assess areas of contamination or PPE breaches	<u>Modifiers</u> <u>Triggers</u> - All actions complete	See Appendix C for details on setting up Glo Germ Aerosolizer




Suspected Coronavirus (COVID-19)

Appendix A: Laboratory Results

None	
------	--


Appendix B: ECGs, X-rays, Ultrasounds and Pictures

Paste in any auxiliary files required for running the session. Don't forget to include their source so you can find them later!

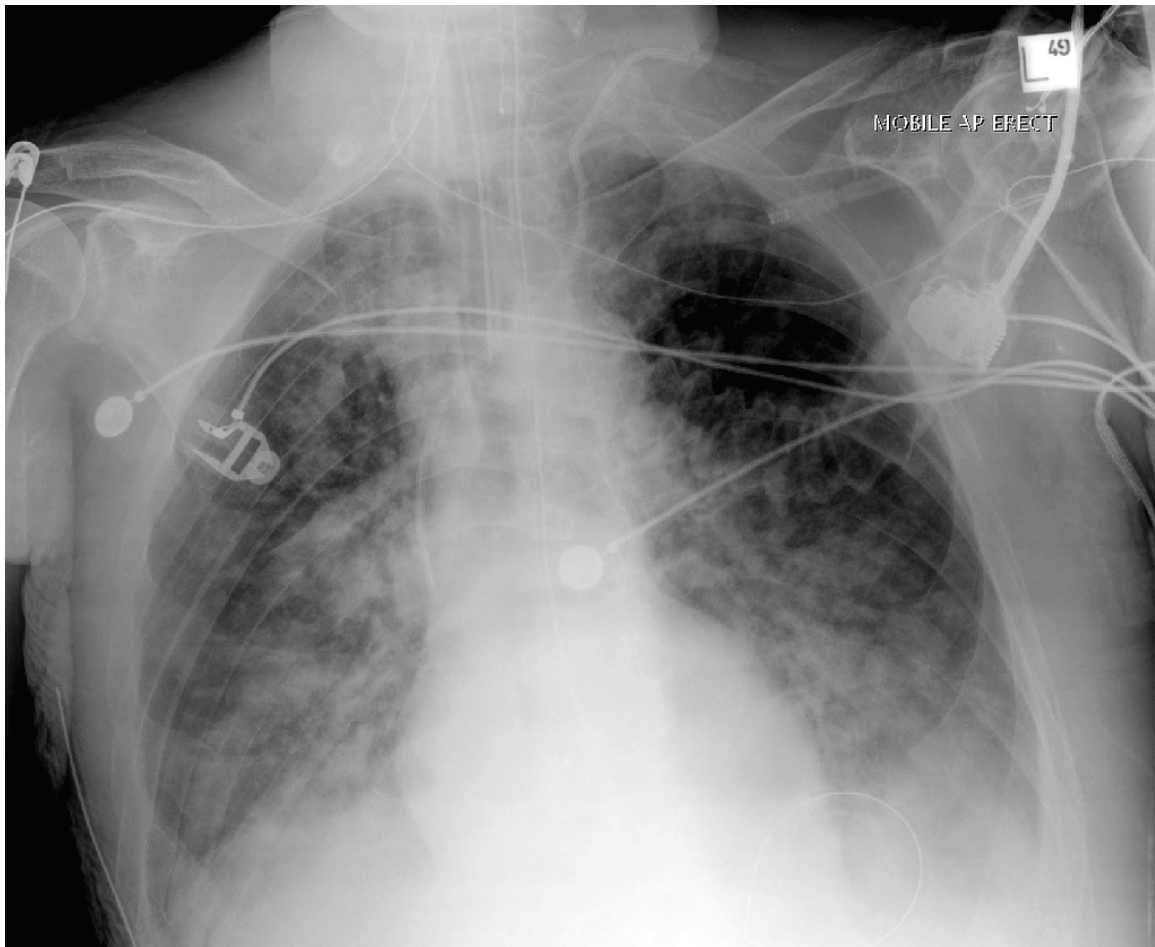


X-ray: Case courtesy of Prof Frank Gaillard, Radiopaedia.org. <https://radiopaedia.org/cases/35985>

Suspected Coronavirus (COVID-19)

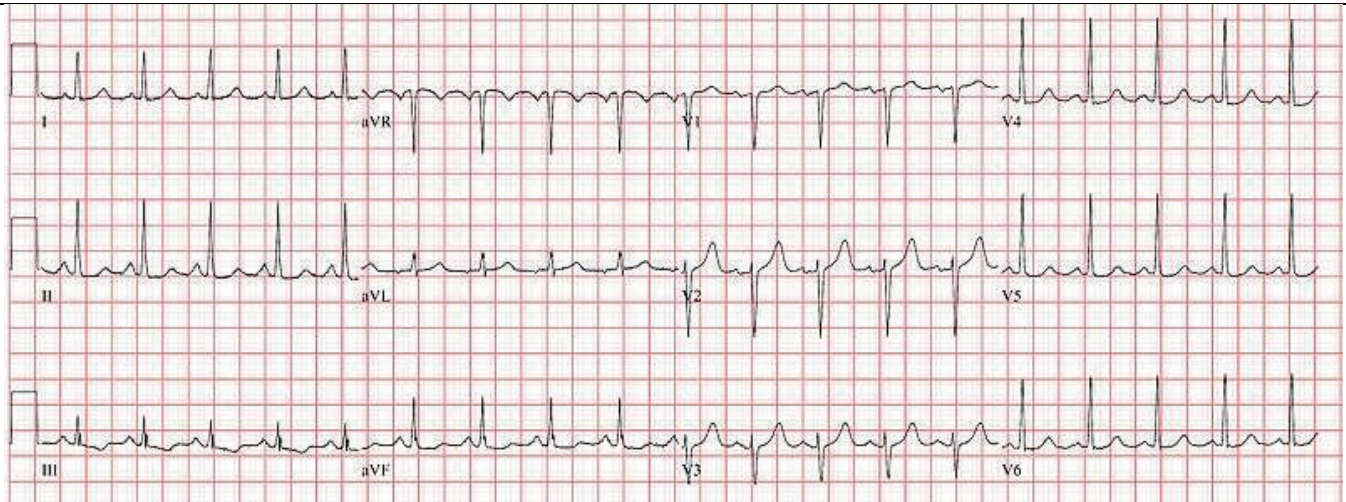

ECG: https://en.ecgpedia.org/wiki/Sinus_Tachycardia

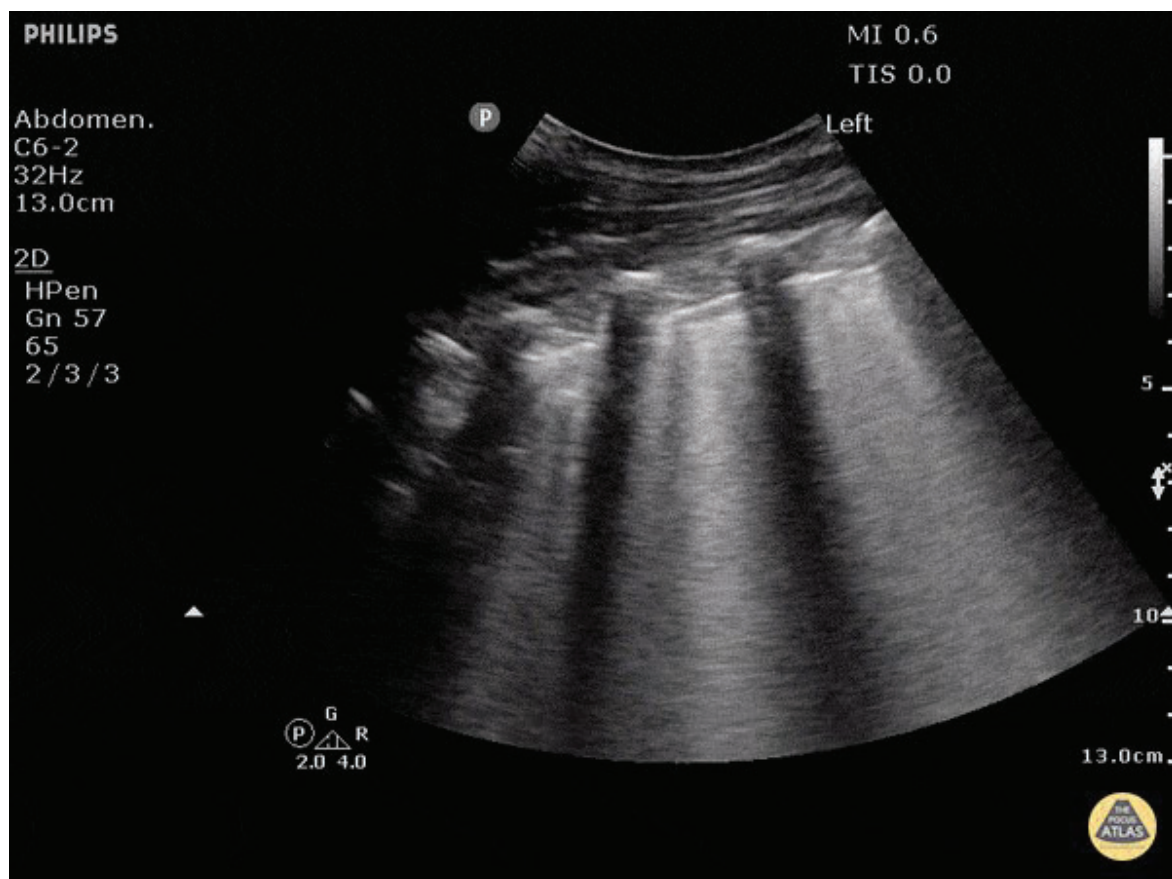

POCUS: <http://www.thepocusatlas.com/pulmonary>

Suspected Coronavirus (COVID-19)

Appendix C: Facilitator Cheat Sheet & Debriefing Tips

In Situ Simulation for Quality Improvement

This case has drawn interest from multiple teams at our site, including teams that are not familiar with the general objectives and principles of in situ simulation. For some observers, this was their first ever experience with in situ medical simulation. A review of basic objectives of in situ simulation is critical to ensure that all participants understand the goals of the exercise.

We suggest hosting a pre-brief and a debrief with the full team and observers, making explicit that in situ simulation is meant to review the current state of our practice, identify latent safety threats, and move towards best practice. Our goal is to **unmask systems factors** that contribute to any breaches in best practice and address them. At our institution, this has led to changes in equipment preparation, standardized tools for communication in isolation rooms, and further educational sessions for staff.

Infection Prevention and Control Considerations

The guidelines for the novel coronavirus are changing frequently as we receive new information about the virus. Infection prevention control (IPAC) considerations also vary between institutions. **Please review the most up-to-date guidelines and discuss with your IPAC team before running the simulation.**

Below are some suggested considerations for review.

- Should a “code resuscitation” be called for these patients, or should the team be kept smaller to mitigate risk to healthcare team/exposure?
- What personal protective equipment (PPE) must providers wear? RT/Intubating MD?
- What mask/PPE should patients be wearing if high risk and being transferred (i.e. from triage to room, to imaging)?
- Who activates IPAC and when?
- What swabs/investigations need to be sent?
- What interventions should be avoided (BiPaP, nebs...)?
- What are next steps for staff members who may have been inadvertently exposed?
- What are the appropriate decontamination measures for equipment (ultrasound)? Should these pieces of equipment not be brought into the room?
- How will we bring equipment (medical supplies, x-ray machine) through the ante room?
- How will the team in the isolation room communicate with the team outside the room?

