



- Soap and water with washcloth for general cleaning.



- Isopropyl alcohol (70%) can be used for disinfection.



- Clorox or Lysol Wipes can be used to disinfect. Please be sure to remove manikin/simulator clothing until fully dry to prevent color transfer from the clothing.



- Purell (or generic) wipes or CaviWipes can be used.

Cleaning & Disinfecting Precautions

AVOID anything that lists kerosene or other fossil fuels as an ingredient! It will eat through the plastic! If you are unsure, call Laerdal Tech Support before use. For extra precaution test in a small inconspicuous area.



Please visit [Laerdal.com](https://www.laerdal.com) and on the homepage is our **COVID-19 Resource Center. Here you will find tools such as:**

- Complimentary Webinars
- Complimentary Scenarios for download
- Complimentary COVID-19 Virtual Patient Case Simulation
- Manikin Cleaning and Hygiene
- Laerdal Medical Devices and COVID-19
- Preparedness Checklist
- Research and Helpful Links
- Respiratory Care Training for COVID-19
- COVID-19 E-Learning Solutions

